

A urinary incontinence device that doesn't attach to the body

Mr. Urinar®

Patent No.4758516, design registration registered

An easy-to-use, comfortable urinary incontinence device developed from an original idea and concept after analyzing the various faults of conventional urinary incontinence devices.

Mr. Urinar provides peace of mind even for incontinence issues.

A strong ally for men who worry about going to the bathroom. Enjoy peace of mind while you're out and about without the device being noticed.


Attaches to your pants for easy attachment and removal!

Insert Mr. Urinar into the fly of your pants and secure it to the pants with the two clips. With a few steps, you can easily attach or remove it while still wearing your clothing.

IT DOESN'T ATTACH DIRECTLY TO THE BODY, SO IT DOESN'T INTERFERE WITH YOUR PENIS OR SKIN.

[Attaching the device]

1 Insert Mr. Urinar into the fly of your pants.

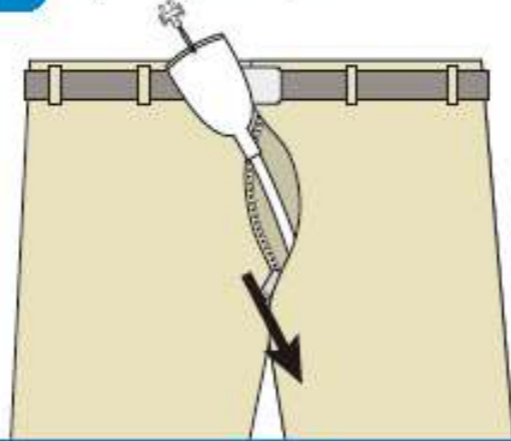

2 Fasten the top clip to the inside of your pants by clipping it to the fabric.



3 Fasten the bottom clip to the area of the pants near the knee by clipping it to the fabric.



Strong, newly designed clips securely attach to clothing

Lightweight urine collection bottle with one-touch attachment and removal

Urine collection tank with a 3D design that forms to the calf and can be easily attached or removed with simple operation.
(Choose from a 250 ml or 350 ml design.)

No stiffness

Comfortable

Lightweight

Two-layer tube construction

Mr. Urinar can be worn while standing or sitting.



Shown with attached

Mr. Urinar - a new urinary incontinence device

A urinary incontinence device that doesn't attach to the body

Mr. Urinar®

Patent No.4758516, design registration registered

Solution to

condom-type and fixed bag-type devices!
disposable underwear and urine collecting pads!

Easy to wear! Affordable!

"I've been waiting for a urinary incontinence device like this."

1 No Discomfort after Urination !

There is no thermal discomfort or itching after urination. In addition, there is no unpleasant stuffiness when wearing the device for a long time and no physical stress from clamping of a belt. You'll enjoy comfort that will make you forget that you're even wearing it. What's more, it doesn't create any skin problems.

2 Large Reduction of the Financial Burden !

Because this device can be washed and used repeatedly, it greatly reduces the financial burden of consumables that are constantly required. It also frees you from having to deal with disposing of disposable underwear.

★ Comparison of financial costs for one month
(When using one Mr. Urinar for six months)



**Monthly savings of 5,500 yen.
Annual savings of 66,000 yen!**

3 Reduced Load on Caregivers !

After urinating, simply remove the bottle with a single touch and dispose of the contents. Clean the inside by adding cleaning solution to the bottle and gently grasp, so little effort is required.

4 A sincere design with innovative ideas !



Easy on the penis

The receiver is made of soft silicone material and has a two-layer cup construction with excellent fit and breathability.



Easy handling

The simple integrated construction can be easily used by anyone. The urine collection tank can be easily attached and removed even when cleaning.



Safe two-layer tube

The tube and nozzle have a specially-designed two-layer construction. The secure design prevents backflow even if the device should fall.

Mr. Urinar can be used for up to six months

※ When worn during the day, not at night, and washed daily

MSRP **¥ 12,000 yen**
(common for different sizes)

target height

S: 150~159 cm

M: 160~169 cm

L: 160~169 cm

Comes with two urine

collection bottles:

250 ml and 350 ml

Mr. Urinar offers reassurance and convenience in various situations



When on a shopping trip



When out for a walk



When stuck in traffic or taking a bus trip



When taking a grandchild to the amusement park

Asahi Industry Co., Ltd

1-11-22, YONBAN, ATSUTA-KU, NAGOYA, AICHI

456-0051 JAPAN

TEL +81-52-671-5193 FAX +81-52-671-5196

E-mail : trading@asahi-sg.co.jp

Website : <http://www.asahi-sg.co.jp>