

A strong ally for men who worry
about going to the bathroom.

MrUrinar®

Patent No.4758516, design registration registered


An easy-to-use,
comfortable urinary incontinence
device developed from an original
idea and concept after analyzing
the various faults of conventional
urinary continence

**IT DOESN'T INTERFERE WITH
YOUR PENIS OR SKIN EASILY**


*Enjoy peace of mind while you're
out and about without the device
being noticed.*

[Attaching the device]


- 1** Insert Mr. Urinar into
the fly of your pants.

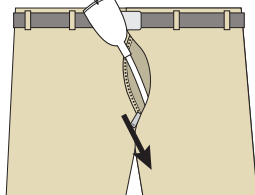

- 2** Fasten the top clip to
the inside of your pants
by clipping it to the fabric.

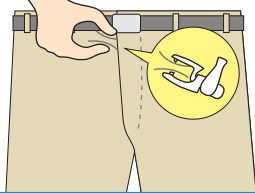

- 3** Fasten the bottom clip
to the area of the pants
near the knee by
clipping it to the fabric.

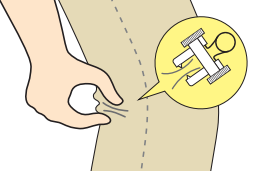

Strong, newly
designed clips securely
attach to clothing

No stuffiness

Comfortable

Lightweight

Two-layer tube
construction

Lightweight urine
collection bottle
with one-touch
attachment and
removal

Urine collection tank
with a 3D design that
forms to the calf and
can be easily attached
or removed with
simple operation.

(Choose from a 250 ml
or 350 ml design.)

Mr. Urinar can be
worn while
standing or sitting.



Shown with attached

With a few steps, you can easily attach or remove it
while still wearing your clothing.

Solution to **dispo-diaper or any pads!** **condom-type or fixed bag-type!**

1 **No Discomfort after Urination!**

There is no thermal discomfort or itching after urination. In addition, there is no unpleasant stuffiness when wearing the device for a long time and no physical stress from clamping of a belt. You'll enjoy comfort that will make you forget that you're even wearing it.

2 **Large Reduction of the Financial Burden!**

Because this device can be washed and used repeatedly, it greatly reduces the financial burden of consumables that are constantly required. It also frees you from having to deal with disposing of disposable underwear.


3 **Reduced Load on Caregivers!**

After urinating, simply remove the bottle with a single touch and dispose of the contents. Clean the inside by adding cleaning solution to the bottle and gently grasp, so little effort is required.

4 **Sincere Design with Innovative Ideas!**



Easy on the penis
The receiver is made of soft silicone material and has a two-layer cup construction with excellent fit and breathability.

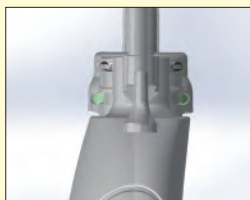

Safe two-layer tube
The tube and nozzle have a specially designed two layer construction. The secure design prevents backflow even if the device should fall.



Easy handling
The simple integrated construction can be easily used by anyone. The urine collection tank can be easily attached and removed even when cleaning.

Size

Small	
Inseam Up to 72 cm	Reference height Up to 160 cm

Length of device

55 cm (with 250-ml bottle attached) /
56.5 cm (with 350-ml bottle attached)

Medium	
Inseam 72 to 77 cm	Reference height 160 to 170 cm


Length of device

60 cm (with 250-ml bottle attached) /
61.5 cm (with 350-ml bottle attached)

Large	
Inseam 77 cm or longer	Reference height 170 cm or taller

Length of device

65 cm (with 250-ml bottle attached) /
66.5 cm (with 350-ml bottle attached)

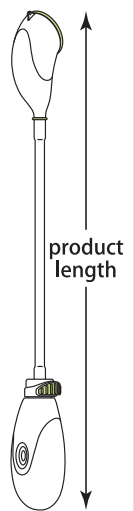

※ Total length of Mr. Urinar does not include the top clip



Comes with two urine collection bottles; 250ml and 350ml

Bottles are small and lightweight and hardly conspicuous while attaching.




Light and small portable urine device.

Mr. Urinar can be used repeatedly by cleaning.

※ When worn during the day, not at night, and washed daily

These options make it easier to use. specialized garter

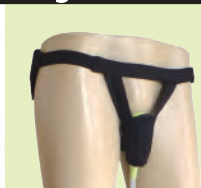

For receiver stability

specialized supporter



For bottle stability

Urine collection bottle 600ml



Mr. Urinar offers reassurance and convenience in various scene!



When on a shopping trip



When out for a walk



When stuck in traffic or taking a bus trip



When taking a grandchild to the amusement park

 **Asahi Industry Co., Ltd.**

1-11-22, YONBAN, ATSUTA-KU, NAGOYA, AICHI

TEL +81-52-671-5193 FAX +81-52-671-5196

E-mail : trading@asahi-sg.co.jp Website : <http://www.asahi-sg.co.jp>