

What's included / Specifications



Product weight: 145g(M size)

Size	I : Inseam H : Reference height L : Product length (with 250ml/350ml bottle)
S size	I : Up to 72cm H : Up to 160cm L : 55cm / 56.5cm
M size	I : 72 to 77cm H : 160 to 170cm L : 60cm / 61.5cm
L size	I : 77cm or longer H : 170cm or taller L : 65cm / 66.5cm

Material

- Receiver : Silicone
- Outer tube : Polyvinyl chloride(PVC)
- Inner tube : Silicone
- Bottle : Polyethylene(PE)
- Cap : Polyacetal(POM)
- Clip (body) : Polystyrene(PS)
- Clip (spring) : Stainless steel (SUS304)
- Special garter : Polyester
- Special supporter : Polypropylene
- Special supporter : Polyester
- Special supporter : Tricot



Men's wearable urine collector AS-URI S/M/L size



Weight: 200g
(Main unit)
Length: 60cm
Pants size: Free size

- Receiver : Silicone
- Outer tube : Polyvinyl chloride(PVC)
- Inner tube : Silicone
- Bottle : Polyethylene(PE)
- Cap : Polyacetal(POM)
- Special pants : Polyester
- Special pants : Cotton
- Special supporter : Nylon
- Special supporter : Polyurethane



Men's wearable urine collector for wheelchair use AS-KUR010




Receiver:
Size: W5.8xD13.5xH6cm
Weight: 70g
Tank:
Size 30xD24xH11cm
Weight: 390g
Capacity: 2.8L
Tube length: 1.5m

- Receiver : Silicone
- Tube : Polyvinyl chloride(PVC)
- Tank : Polyethylene(PE)
- Cap : Polyacetal(POM)
- Membrane : Latex
- Special garter : Polyester



Men's wearable urine collector for sleeping position AS-DUR010

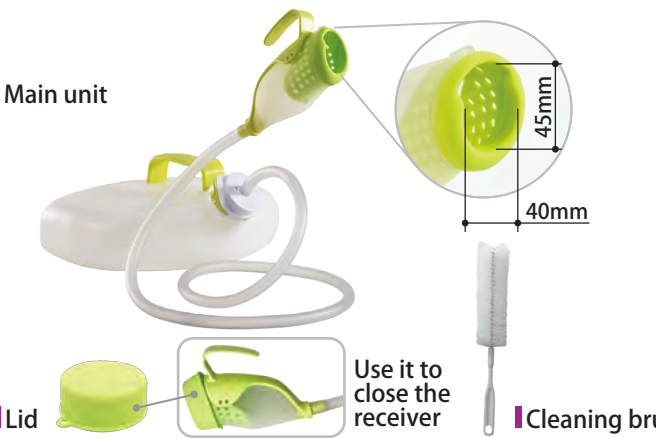

Receiver:
Size: W5.8xD13.5xH6cm
Weight: 70g
Tank:
Size: W30xD24xH11cm
Weight: 390g
Capacity: 2.8L
Tube length: 1.5m

- Receiver : Silicone
- Tube : Polyvinyl chloride(PVC)
- Tank : Polyethylene(PE)
- Cap : Polyacetal(POM)
- Handle : Polyacetal(POM)
- Lid : Silicone



Men's hand-held urine collector AS-TUR010

Men's urine collector

- Soft and comfortable silicone type
- Lightweight and prevents moist stuffy feeling
- Washable and reusable
- Reduces burden on caregivers

Men's urine collector made of soft silicone

A wearable urinary collector for those who worry about having to go all the way to the bathroom. It is inconspicuous even when worn, so you can go outside with peace of mind.



Wearable type

AS-URI S/M/L

A urinary collector made specifically for wheelchairs that was developed for wheelchair users. It reduces the anxiety and burden of having to go all the way to the bathroom, and you can go outside with a peace of mind.



Wearable type for wheelchair use

AS-KUR010

Wearable urinary collector for sleeping position. You can wear it to bed.



Wearable type for sleeping position

AS-DUR010

A piece of mind that you do not need to go all the way to the bathroom. You can urinate on your own even in the middle of the night.



Hand-held type

AS-TUR010



Recommended users

Product / Features

How to use

- For those who have problems with urinary incontinence
- For those who worry about a sudden urge to urinate
- Can be used in both standing and sitting positions
- For indoor/outdoor use
You can wear it when going outdoors.

The double-sheaved receiver is made of soft silicone that is breathable and gentle to the skin.

Receiver, Inner receiver, Tube, Double-sheathed tube, Clip, Bottle, Cap

With a double-sheathed tube, the flow is smooth.

Simply detach the bottle with a single touch and dispose of the contents.

Wearable type

*Since it is worn under the pants, it cannot be seen from the outside.

- Attach the receiver to the special garter.
- Adjust the height and left / right position with the waist belt.
- Pass the special supporter between your legs and secure it.
- Put the urine collection bottle in your pocket and secure it.
- Wear your pants over it.
- Please make sure the flow track is unobstructed so that urine can flow smoothly during use.
- You need to make sure it is flowing downward naturally.
- After use, please wash and disinfect daily and let it dry.

AS-URL S/M/L

- For those who have problems with urinary incontinence
- For those who worry about a sudden urge to urinate
- Can use while in a wheelchair
- For indoor/outdoor use
You can wear it when going outdoors.

The double-sheaved receiver is made of soft silicone that is breathable and gentle to the skin.

Receiver, Inner receiver, Tube, Double-sheathed tube, Clip, Bottle, Cap

With a double-sheathed tube, the flow is smooth.

Simply detach the bottle with a single touch and dispose of the contents.

Wearable type for wheelchair use

*Since it is worn under the pants, it cannot be seen from the outside.

- Secure it so that the special pants wrap around the receiver.
- Secure the pants with the Velcro.
- Put the urine collection bottle in your pocket and secure it.
- Wear your pants over it.
- Please make sure the flow track is unobstructed so that urine can flow smoothly during use.
- You need to make sure it is flowing downward naturally.
- After use, please wash and disinfect daily and let it dry.

AS-KUR010

- For those who want to be able to urinate in a sleeping position
- You can wear it in a sleeping position.
- For indoor use

The double-sheaved receiver is made of soft silicone that is breathable and gentle to the skin.

Receiver, Inner receiver, Tube, Double-sheathed tube, Clip, Bottle, Cap, Urine collection tank, Backflow prevention membrane for peace of mind, Inside the receiver, Easy flow shape

Simply detach the tank with a single touch and dispose of the contents.

Wearable type for sleeping position

Use it at an angle for easy urine flow.

Depending on the situation, you may need to make an opening in the underwear or pants to allow the hose to pass through, making it easier for urine to flow.

- Attach the receiver to the special garter.
- Adjust the height and left / right position with the waist belt.
- Wear your pants over it.
- Please make sure the flow track is unobstructed so that urine can flow smoothly during use.
- You need to make sure it is flowing downward naturally.
- Please wash and disinfect daily and let it dry.

AS-DUR010

- For those who want to use it at the bedside, etc.
- You can urinate without having to go all the way to the bathroom.
- It can be used in standing, sitting and supine positions
- For indoor use

The double-sheaved receiver is made of soft silicone that is breathable and gentle to the skin.

Receiver, Inner receiver, Tube, Double-sheathed tube, Clip, Bottle, Cap, Urine collection tank, Handle shape, Inside the receiver, Easy flow shape

Simply detach the tank with a single touch and dispose of the contents.

Hand-held type

Face-up position, On-one-side position, Sitting upright position, Knee gatch position, (next to bed) Standing position, (in wheelchair) Sitting position

- Please make sure the flow track is unobstructed so that urine can flow smoothly during use.
- You need to make sure it is flowing downward naturally.
- Please wash and disinfect daily and let it dry.

AS-TUR010